

Additional Information on the Medical Device
HEALER® ANTISEPTIC SOLUTION WITH RASPBERRY FRAGRANCE, 1000 ml

Intended Purpose

HEALER® Antiseptic Solution is a medical device intended for cleansing and antiseptic treatment of the skin surrounding superficial injuries, postoperative sutures and superficial first- and second-degree burns, as well as minor skin injuries limited to the epidermis (abrasions, scratches) that do not require aseptic conditions, in order to reduce the microbial load and prevent secondary contamination.

The device is also used for hygienic and antiseptic treatment of intact skin of the hands of patients and healthcare personnel before non-invasive medical procedures and before putting on medical gloves.

The solution exhibits antimicrobial activity against transient and resident skin microflora and helps suppress the growth of microorganisms until the first washing of the skin.

Auxiliary components provide gentle skin care, help maintain the natural level of skin hydration and its physiological condition.

Indications for Use

- superficial skin injuries (abrasions, scratches);
- skin surrounding wounds and postoperative sutures;
- superficial first- and second-degree burns;
- antiseptic treatment of intact skin before non-invasive medical procedures;
- hygienic hand treatment for healthcare personnel;
- care of the skin of the feet and nail folds as an auxiliary hygienic measure.

Directions for Use

Hygienic Hand Treatment

Apply at least 3 ml of the solution to clean, dry, intact skin of the hands, distribute evenly over the entire surface and rub in for at least 30 seconds. Allow to dry naturally. Medical gloves should only be put on after the hands are completely dry.

Superficial Injuries and First- and Second-Degree Burns

Spray the affected area or treat it with a clean disposable gauze swab soaked in the solution. Ensure complete wetting of the surface, leave for at least 30 seconds, then allow to dry naturally. Repeat the treatment if necessary.

Contraindications

Hypersensitivity to any component of the device.

Contact with the mucous membrane of the eyes.

Use in the middle or inner ear.

Use in patients with lesions of the central nervous system.

Do not use simultaneously with anionic surfactants.

Warnings

The device is non-sterile.

Do not apply directly to deep, bleeding or infected wounds.

For external use only.

Do not use for injection or infusion.

Avoid contact with the eyes.

Adverse Reactions

In rare cases, allergic reactions may occur. If signs of individual hypersensitivity appear, discontinue use of the device.

Restrictions

Use during pregnancy, breastfeeding and in young children is recommended only when the expected benefit outweighs the potential risks.

Composition

Polyhexamethylene biguanide hydrochloride (PHMB), glycerin, polyethylene glycol, purified water.

Presentation

Transparent polymer bottle 60 ml with spray pump and cap;

PE canister 1000 ml with cap or dispenser;

PE canister 5000 ml with cap.

Storage Conditions

Store in a place protected from light and out of the reach of children at a temperature between +2 °C and +35 °C.

Close the container tightly after use to prevent contamination of the contents.

Avoid contamination of the bottle neck.

Shelf Life

3 years from the date of manufacture.

The manufacturing date corresponds to the batch number.

Manufacturer**HEALER LLC**

1/33-A Politekhnichnyi Lane,

Kyiv, 03055,

Ukraine

Tel.: +380 800 33 31 31

E-mail: info@healer.bio

Website: www.healer.ua

Date of the last revision of the information: 20 October 2025



Producer



Batch number



Use up to



UA.TR.098

Mark of conformity
to technical
regulations